



Two-Panel 6" Surgical Rib Belts
For Men & Women

WOMEN'S
 Model No. 34-4200 

MEN'S
 Model No. 34-4100

Size: Universal 



For weakened muscles and soft tissues of the rib cage as a result of:



- Strain
- Overuse
- Rib Fracture
- Sternum Fracture

Premium Woven Series - Latex-Free, Surgical Paneled

Paneled RIB BELTS

for Men & Women

Help stabilize rib and sternum fractures by limiting expansion through compression. They provide comfort and encourage more controlled breathing to help reduce pain. Rib belts also provide support and compression to the muscles and soft tissues of the rib cage weakened by strain, trauma or overuse, injury or surgery.

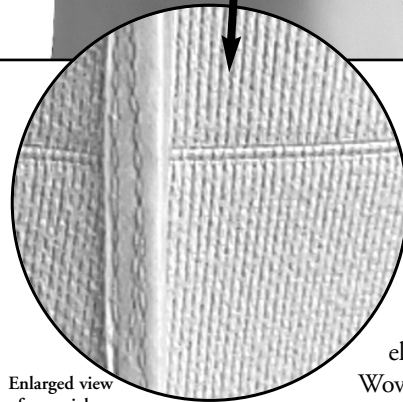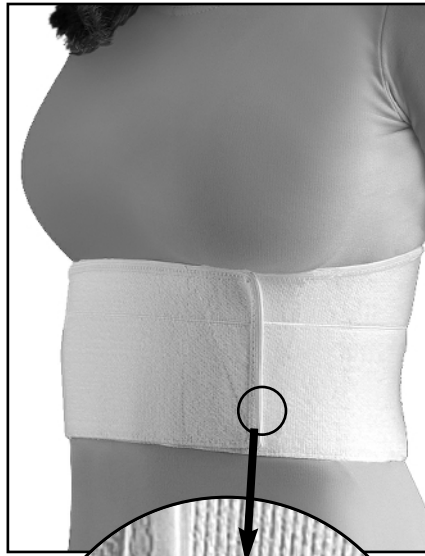

Enlarged view of material.

PREMIUM WOVEN-SERIES LATEX-FREE, SURGICAL ELASTIC

TRUE WOVEN ELASTIC – Woven, as opposed to knit elastic, is the premium construction in elastic materials. Woven elastic gives firm and consistent compression power and can be cut without unraveling.

PANELED HINGED STITCH – This innovative design has a simulated hinged stitch that creates separate panels for a more conforming, tapered fit with flexibility. Old style panel rib belts were connected with an open, crochet-like stitch that easily pinched the skin and left an opening for swelling pool. FLA's unique simulated hinged stitch eliminates these problems.

HOOK COMPATIBLE EXTERIOR – Allows for a wider range of adjustability and easy application. Holds up through repeated use.

LATEX FREE AND PLUSH LINING – Comfortable and non-irritating for patients and caregivers with sensitivity to latex rubber.

CARE INSTRUCTIONS:

Fasten closure and hand wash in cold water with mild soap. Wrap in towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING:

Measure around chest at base of sternum.

Size	Fits
Universal	24 - 28"


800-327-4110

**Two-Panel 6" Surgical
Rib Belts**
For Men & Women

WOMEN'S
Model No. 34-4200

MEN'S
Model No. 34-4100

Size: Universal

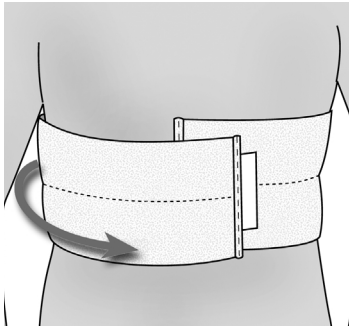

For weakened muscles and soft tissues of the rib cage as a result of:

- Strain
- Overuse
- Rib Fracture
- Sternum Fracture



800-327-4110

Premium Woven Series - Latex-Free, Surgical Paneled



**Paneled
RIB BELTS**
for Men & Women



**APPLICATION
INSTRUCTIONS**

Ensure the product is comfortable to wear and giving the proper compression. Properly stretching the binder and setting the desired compression will not restrict breathing. It will help to facilitate deep breathing.

INDIVIDUAL APPLICATION:

Easy and inconspicuous to wear under most clothing.

1. Apply in a standing or laying position.
2. Open belt to its fullest length, with the care tag on the inside.
3. Position the rib belt around the rib cage. The hook should be on the right side
4. Stretch both sides and wrap the side without the hook around the body first.
5. Secure the hook onto the support where needed.
6. Adjust as necessary.

CAREGIVER APPLICATION:

1. With patient lying flat on his/her back in bed, open the binder to its fullest length, positioning it underneath the patient with the plush side against the patient's skin.
2. Position the rib belt around the rib cage. The hook should be on the right side of the patient.
3. Stretch the left side of the binder and pull it across the patient's ribs/abdomen, using the weight of the patient to hold it in place.
4. Holding the left side in place on the patient's ribs/abdomen, stretch the right side the same way securing to the loop.
5. Adjust as necessary.