



MULTI AFO/ CONTRACTURE SPLINT

Model No: 58-100

Sizes: Small - X-Large
Color: White



with Transfer Pad
Model No: 58-120

Sizes: Small - X-Large
Color: White

INDICATIONS:

- Non-Fixed Contractures
- Foot Drop
- Pressure Ulcers
- Ankle Deformity
- Knee Flexion
- Hip Abduction
- Post Knee or Hip Surgery
- Hip or Leg Rotation

800-327-4110

PRODUCT FEATURES:

- **Unique Splint Shape** – Maintains functional alignment of the foot and ankle
- **Lightweight Plastic Shell** – Has a spring-like action that yields to the contracture
- **Microban® Anti-Microbial Protection** – Liner is treated with Microban® for a fresher, more hygienic product.
- **Soft and Comfortable Liner** – Allows air to circulate, absorbs perspiration, and also keeps the foot warm.
- **Heel Suspension** – Protects the heel from pressure or friction
- **Hip Rotation Control Bar** – Helps position the foot correctly by preventing internal or external rotation of the hip, foot and/or ankle
- **Ankle Strap** – Reinforces foot position
- **Adjustable Padded Toe Post** – Protects the toes from contact with bed linens
- **Removable Ambulation Pad** (*Model 25-120 only*) – Assists the patient in transfers, i.e. from bed to chair, and also in therapy treatment plan when walking is necessary.



HOW TO APPLY:

Unfasten all closures (calf strap, ankle strap and foot strap). Gently place the patient's knee in a bent position and slide the splint in place. While applying pressure over the instep, firmly position the foot in the contracture splint. Maintaining pressure over the instep, fold the liner and secure hook and loop closure over the instep. Secure hook and loop closure around the calf. Secure ankle strap.

CARE INSTRUCTIONS:

For Liner: Fasten all closures. Machine wash on gentle cycle or hand wash in cold water with a mild soap. Rinse thoroughly. Wrap in a towel to remove excess water. Do not wring or twist. Air dry away from heat or tumble dry on cool. Do not bleach. The Microban® Antimicrobial Protection is part of the liner's molecular structure and will not wash out through normal laundering of the liner. Microban® protection should not replace or eliminate your normal use and care or laundering procedures. Liner should be laundered one a week or more frequently if needed. **For Plastic Shell:** Spot clean with a soft cloth and mild soap. Rinse and wipe dry.

SIZING:

Measure according to US shoe size.
Fits Right or Left.

Size	Men's Shoe	Women's Shoe	Calf Size
Small	4 - 5	3 - 6	12 - 16"
Medium	6 - 9	7 - 10	14 - 18"
Large	10	11	16 - 20"
X-Large	Over 10	Over 11	18 - 22"



Website: **www.flaorthopedics.com**

©2004 FLA ORTHOPEDICS, INC.